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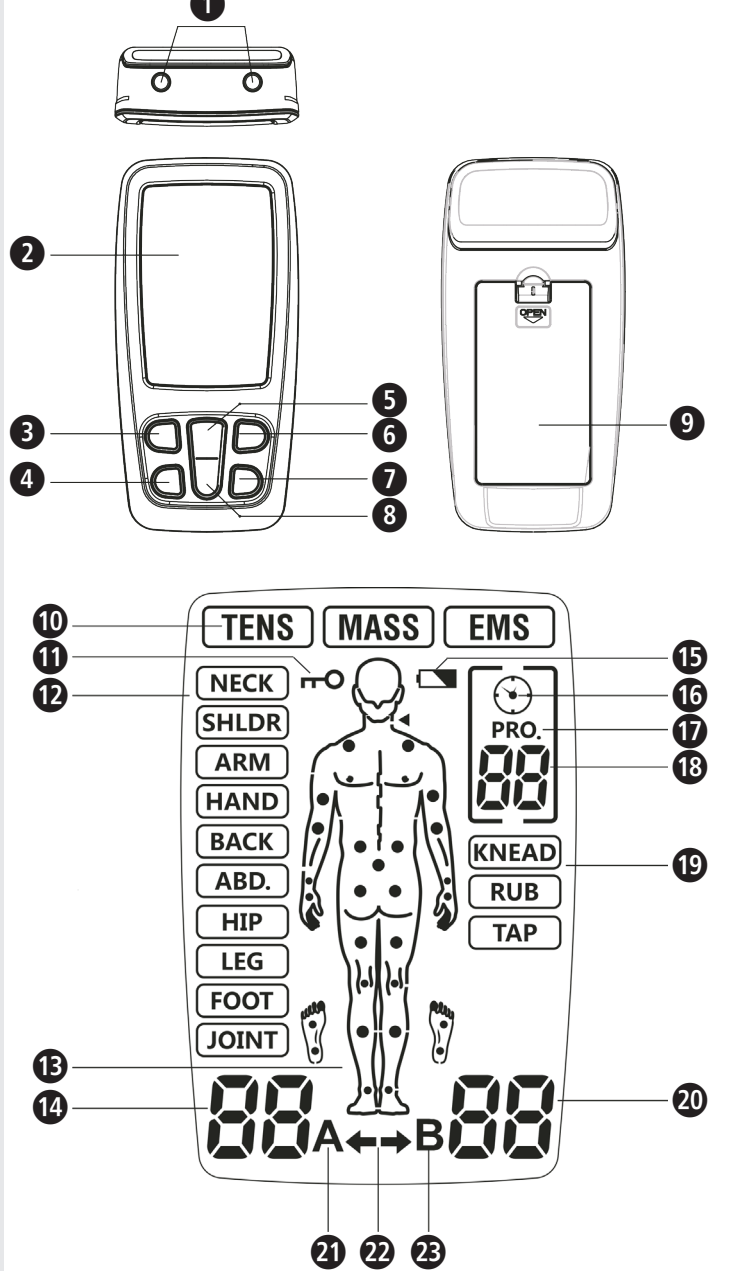
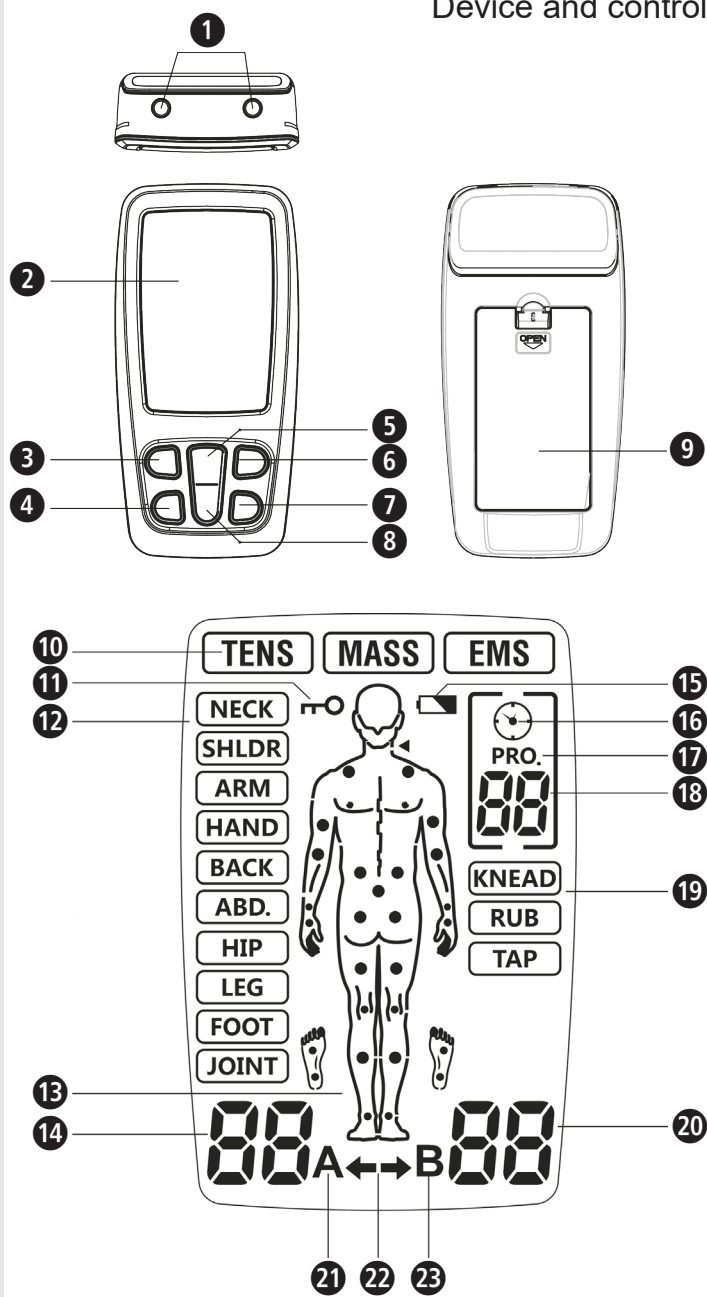
Device and controls

Explanation of symbols

**IMPORTANT**  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device.

**NOTE**  
These notes give you useful additional information on the installation or operation.

Device classification: Type BF

LOT LOT number

IP22 Protection rating against foreign objects and water

Manufacturer

Date of manufacture

GB Safety instructions

**Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.**

**Intended use**  
The medisana 3-in-1-Electro Therapy Device TT 200 is intended to be used for the following applications:

- **TENS Mode:** It is used for temporary relief of pain associated with sore and aching muscles in the neck, shoulder, back, joint, hip, hand, abdomen, foot, upper extremities (arm) and lower extremities (leg) due to strain from exercise or normal household work activities.
- **EMS Mode:** The EMS stimulation program stimulates healthy muscles in order to improve and facilitate muscle performance.
- **Massage mode:** The Massage stimulation program provides relaxing muscle vibration to loosen tight muscles.

**Contraindications**

- Do not use this device if you are using a cardiac pacemaker, implanted defibrillator or other implanted metallic or electronic devices. Such use could cause electric shock, burns, electrical interference, or death.
- The device should not be used when cancerous lesions or other lesions are present in the treatment area.
- Stimulation should not be applied over swollen, infected, inflamed areas or skin eruptions (e.g. phlebitis, thrombophlebitis, varicose veins, etc.).
- Electrode placements must be avoided in the carotid sinus area (anterior neck) or transcranially (through the head).
- This device should not be used in overly enervated areas.
- In the case of an existing inguinal hernia, the device must not be used.
- Do not use on scarred areas following a surgery for at least 10 months after the operation.
- Do not use with serious arterial circulatory problems in the lower limbs.

- Therapeutic sessions with the medisana TT 200 do not replace medical diagnosis or treatment. Ask your doctor about each kind of pain or disease, before using the device.
- If your pain does not diminish, does not become significantly milder or last longer than 5 days, discontinue use and contact your doctor.
- Do not apply stimulation over your front and lateral neck because this could cause severe muscle spasms resulting in closure of your airway, difficulty in breathing or adverse effects on heart rhythm or blood pressure.
- Do not apply stimulation across your chest because the application of electrical current to the chest may cause rhythm disturbances to your heart, which could be lethal.
- Do not apply stimulation over - or in proximity to - cancerous lesions.
- Do not apply stimulation in the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), which may not operate properly when electrical stimulation device is in use.
- Do not apply stimulation when in bath or shower.
- Do not apply stimulation while sleeping.
- Do not apply stimulation while driving, operating machinery or during any activity when electrical stimulation can put you at risk of injury.
- Apply stimulation only to normal, intact, clean, healthy skin.
- The long-term effects of electrical stimulation are unknown. Electrical stimulation device cannot replace drugs.
- Stimulation should not take place while the user is connected to high-frequency surgical equipment, which may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator.
- Do not use the device in the vicinity of short or microwave therapeutic devices as they may affect the output power of the device.
- Never use it near the cardiac area. Stimulation electrodes should never be placed anywhere on the front of the thorax (marked by ribs and breastbone), but above all not on the two large pectoral muscles. There it can increase the risk of ventricular fibrillation and lead to cardiac arrest.
- Never use it on the eye, head and face area.
- Never use it near the genitals.
- Never use it on the areas of the skin which lack normal sensation.
- Keep electrodes separated during treatment. It could result in improper stimulation or skin burns if electrodes are in contact with each other.
- Keep the stimulator out of reach of children.
- The appliance is intended for use by adults only.
- The therapy should be pleasant. If you feel pain or feel the application is unpleasant, do not increase the intensity but stop the usage and consult your doctor.
- TENS is not effective for pain of central origin including headache. It is not a substitute for pain medications and other pain management therapies.
- TENS is a symptomatic treatment and, as such, suppresses the sensation of pain that would otherwise serve as a protective mechanism.
- Effectiveness is highly dependent upon the patient selection for a practitioner qualified in the management of pain patients.
- Since the effects of stimulation of the brain are unknown, stimulation should not be applied across your head, and electrodes should not be placed on opposite sides of your head.
- The safety of electrical stimulation during pregnancy has not been established.
- You may experience skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive medium (silica gel).
- If you have suspected or diagnosed heart disease or epilepsy, you should follow precautions recommended by your physician.
- Use with caution if you have a tendency to bleed internally, e.g. following an injury of fracture or during your menstrual period.
- Consult with your physician prior to use the device after a recent surgical procedure, because stimulation may disrupt the healing process.
- For single patient use only.
- This stimulator should not be used by patients who is noncompliant and emotionally disturbed including dementia.
- Only use the device as described in this instruction manual. Any other use will be deemed as not in accordance with the instructions and may lead to damage to property or even injury to persons. medisana assumes no liability for damages incurred as the result of improper use. The device is designed for private use.
- Rare cases of skin irritation may occur at the site of the electrode placement following long-term application.
- Do not use this device in the presence of other equipment which sends electrical pulses to your body.
- Do not use sharp objects such as a pencil or ballpoint tip to operate the buttons on the control panel.
- Check the electrode connections before each use.
- Electrical stimulators should be used only with the electrodes recommended for use by the manufacturer.

Adverse Reactions

- Possible skin irritation or electrode burn under the electrodes may occur.
- On very rare occasions, first-time users of EMS report feeling light-headed or faint. We recommend that you use the product while seated until you become accustomed to the sensation.
- If the stimulation makes you uncomfortable, reduce the stimulation intensity to a comfortable level and contact your physician if problems persist.

Scope of supply

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the it and contact your dealer or your service centre.

The following parts are included:

- 1 medisana 3-in-1 Electro Therapy Device TT 200
- 4 Electrodes 50x50 mm
- 2 Connection cables
- 3 Batteries AAA 1,5V
- 1 Instruction manual

If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**  
Please ensure that polythene packing is kept away from the reach of children!  
Risk of suffocation!

Device and controls

- ① Output socket
- ② Display
- ③ On-/Off-/M-Button
- ④ T-Button
- ⑤ + Button
- ⑥ B-Button
- ⑦ CH-Button
- ⑧ - Button
- ⑨ Battery compartment cover
- ⑩ Therapy mode
- ⑪ Device locked
- ⑫ Body region
- ⑬ Recommended treatment points
- ⑭ Intensity channel A
- ⑮ Battery low
- ⑯ Timer symbol
- ⑰ Program symbol
- ⑱ Program no. / Therapy time
- ⑲ Massage mode
- ⑳ Intensity channel B
- ㉑ Symbol channel A
- ㉒ Channel indicator
- ㉓ Symbol channel B

Insert / replace batteries

Open the cover of the battery compartment on the back of the device and insert the enclosed 3 batteries AAA 1.5V according to the symbols (+/-). Close the battery compartment again. When changing the batteries later on, only use 1.5V batteries of type AAA (Micro/LR6).

SAFETY NOTES FOR BATTERIES

- Do not disassemble batteries!
- Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.
- Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes!
- If battery acid comes in contact with the skin, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed, seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Keep batteries out of children's reach!
- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Cable connection

Ensure that the device is completely switched OFF before connecting the cables. First connect the electrodes with the two connecting cables and then the cables with the device. On the upper side of the device there are two output sockets ① for the connection of the electrode lines. The device has two connections for two channels (A and B). You may decide according to your personal needs, if you want to use only one channel with one pair of electrodes or both channels with two pairs of electrodes. With two pairs of electrodes you can simultaneously stimulate two different body regions.

**WARNING**  
Do not insert the plug of the electrode wires into any AC power supply socket!

Applying the electrodes

The re-usable protection foil may be pulled off directly before usage - keep it for later use. Now place the electrodes according to your pain area (following the instructions of your doctor).

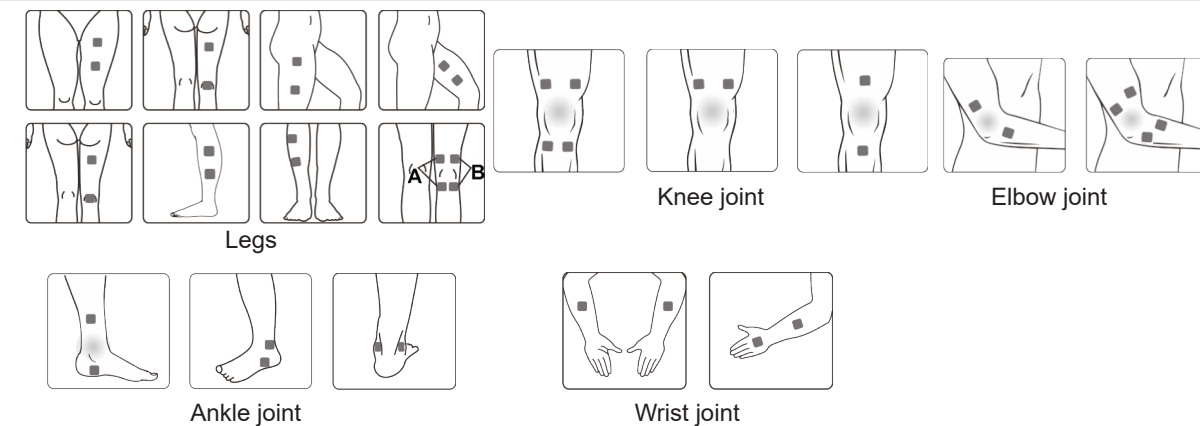
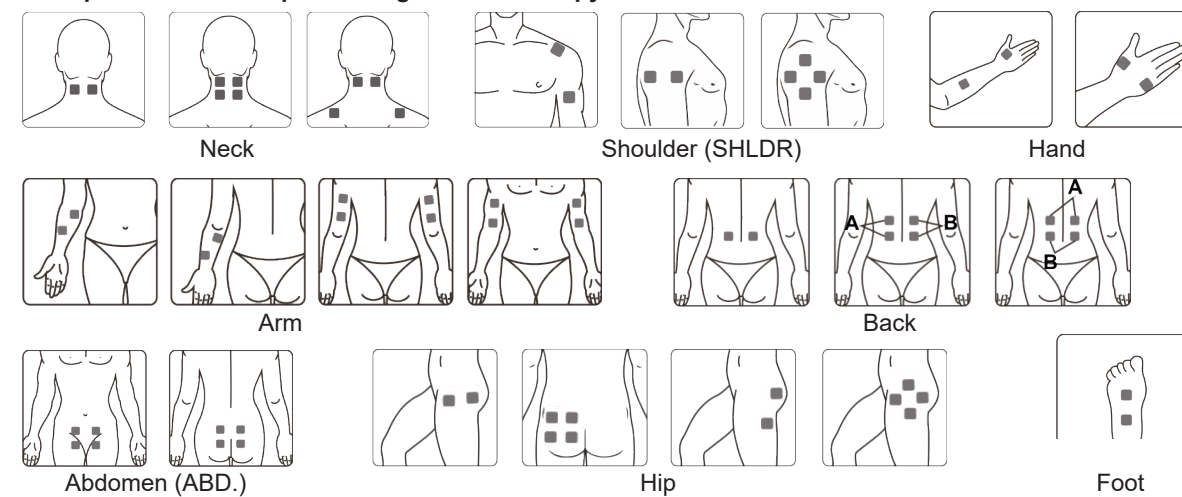
**WARNING**

- The device must be deactivated before applying the electrodes!
- Before applying the electrodes, remove residual skin cream or ointment. The skin has to be dry and free of fat.
- Do not place the electrodes on injured or inflamed skin areas (wounds, pimples, skin rash, reddening etc.).
- Always place the electrodes according to the doctor's instructions before you switch on the device.
- Make sure that the electrodes cannot fall off during therapy. This can cause an unintentional electric shock!
- Switch off the device immediately, if an electrode comes off the skin during treatment.


Below are placement suggestions for the electrodes depending on the therapy mode. Keep in mind, however, that each patient responds differently to stimulation therapies - so positioning the electrodes and choosing a program that meets your needs may differ from the therapies presented here. The optimal parameters for you (electrode position, program and intensity) should be used for the following treatments again, so write them down to remember.

For each program, the recommended treatment points ⑬ are displayed below. The following figures show an example of attaching the electrodes at these treatment points.

Examples of electrode positioning in TENS therapy mode:

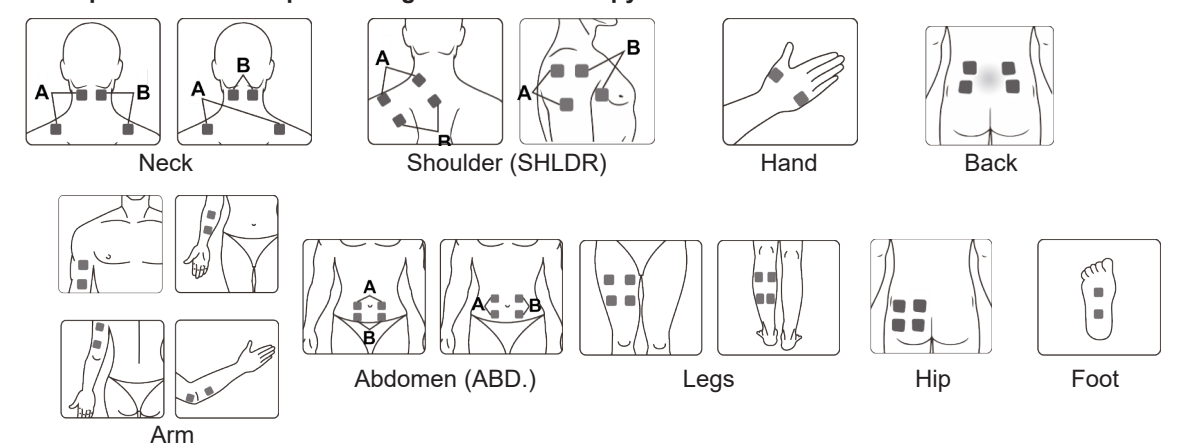Examples of electrode positioning in the EMS therapy mode:


Switch on the device

To switch on the device, shortly press the On-/Off-/M-Button ③. The display will be activated and the device will switch into standby mode. Now choose the therapy mode with the On-/Off-/M-Button ③:


TENS : to fight pain (TENS = Transcutaneous Electrical Nerve Stimulation)

MASS : Massage to relax tense muscles

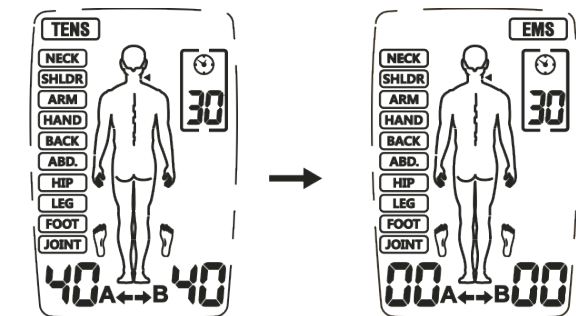EMS : to build up the muscles (EMS = Electrical Muscle Stimulation)

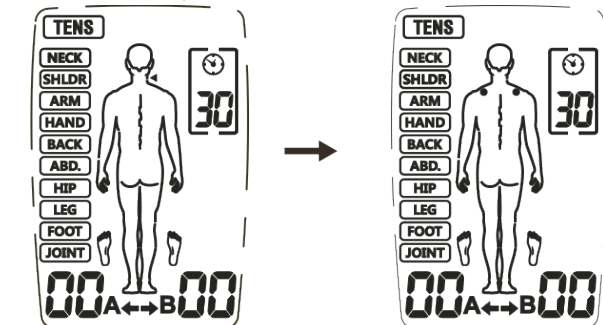
Then use the B-button ⑥ to select the body area to be stimulated:


Now choose the channel (A, B or both channels) with the CH-button ⑦:

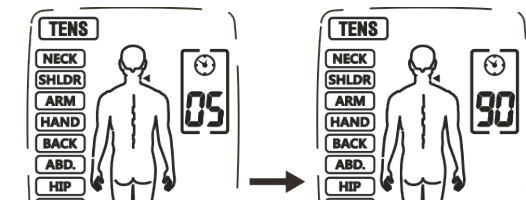Adjust the treatment time with the T-button ④:

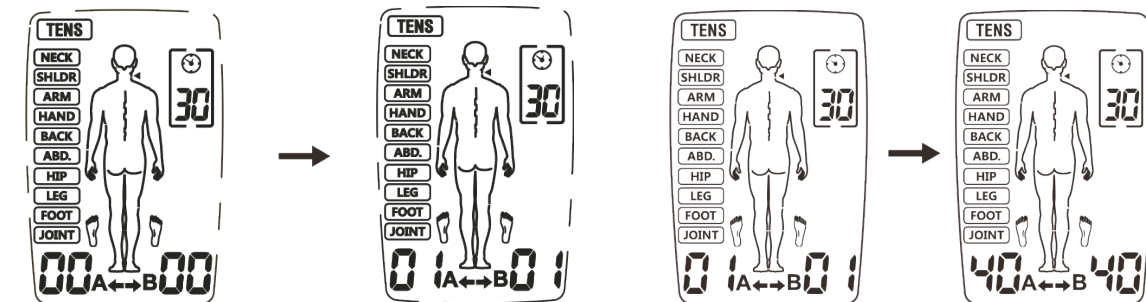
Now press the + button ⑤ to increase the intensity of the selected channel. The display will show the corresponding intensity level. You may increase the intensity up to the maximum level of 40 by repeatedly pressing the + button ⑤:

You can decrease the intensity by pressing the - button ⑧. Decrease the intensity to level 0 and the device will switch to standby mode.

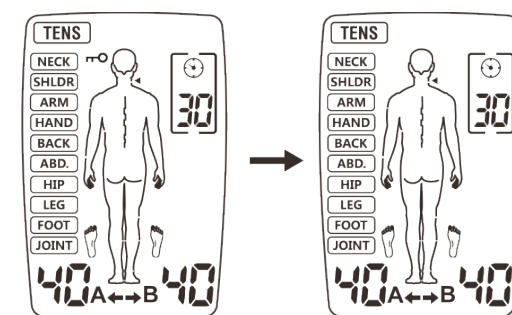
During the treatment, press and hold B-button ⑥ to turn on the lock function. The symbol ① will appear on the display. When the lock function is active, you cannot adjust the intensity inadvertently. To switch off the lock function, press and hold the B-button ⑥, until the symbol ① disappears.

## GB Instruction manual 3-in-1 Electro Therapy Device TT 200
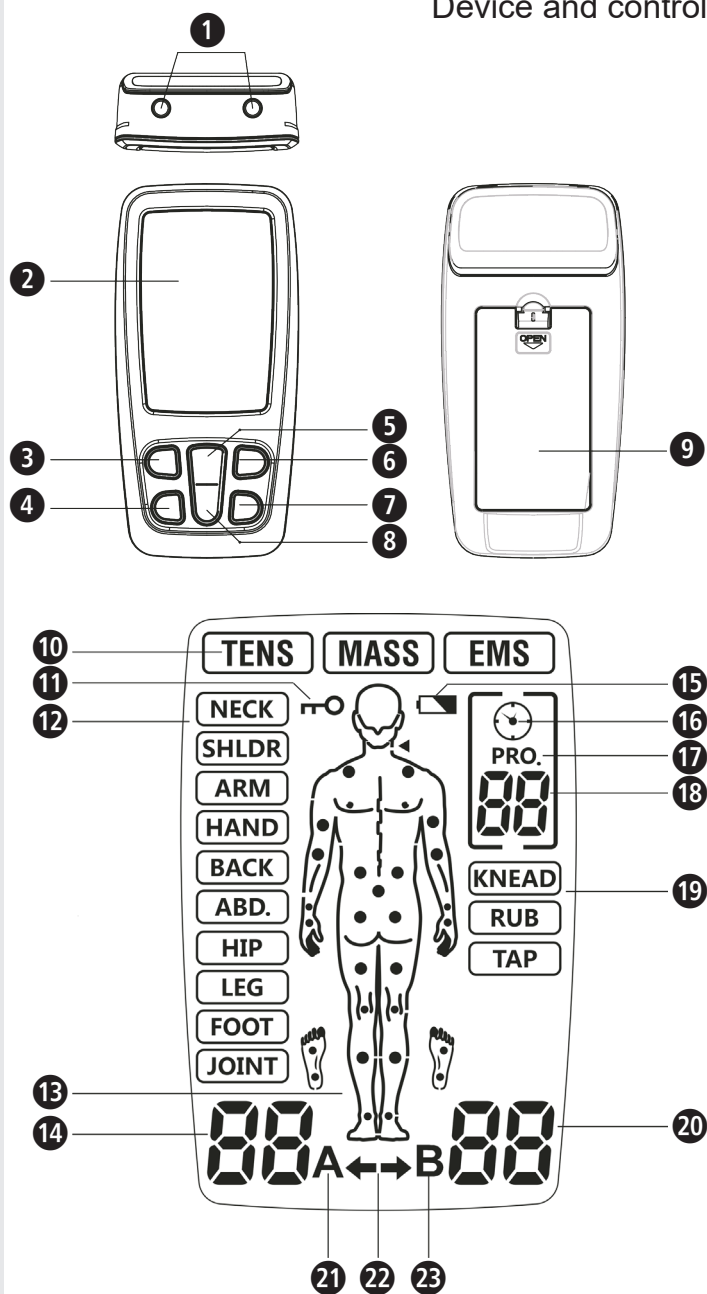
### Device and controls

### Explanation of symbols

#### IMPORTANT

Non-observance of these instructions can result in serious injury or damage to the device.

#### WARNING

These warning notes must be observed to prevent any injury to the user.

#### CAUTION

These notes must be observed to prevent any damage to the device.

#### NOTE

These notes give you useful additional information on the installation or operation.

Device classification: Type BF

LOT number

Protection rating against foreign objects and water

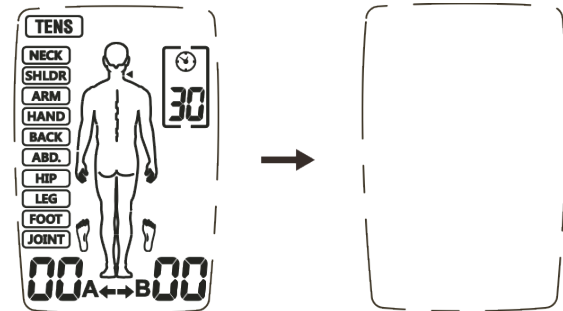
Manufacturer


Date of manufacture

Press the On-/Off-/M-Button to end the treatment. Pressing the On-/Off-/M-Button again switches the device off.

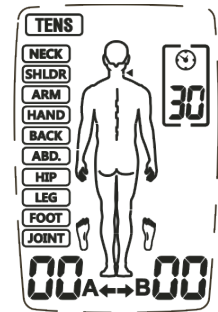The device has an automatic load detection at all set intensities above level 5. If the electrodes are not properly applied to the skin or another connection problem appears, the device will automatically switch to level 0, while the symbol „A“ or „B“ flashes. The device then switches back to standby mode.

If the battery voltage is too low, the „low battery“ symbol flashes. Stop the application and replace the batteries.

#### Use of the electrodes

- The supplied electrodes may only be used with the medisana TT 200 3-in-1 electro-therapy device. Make sure that the device is switched off before applying and removing the electrodes.
- If you want to reposition the electrodes during the application, turn the device off first.
- The usage of electrodes may lead to skin irritations. If you experience such skin irritations, e.g. redness, blistering or itching, discontinue using them. Do not use the stimulator permanently on the same body part, as this may also lead to skin irritations.
- Never use the electrodes together with other people.
- The electrodes must connect entirely to the skin surface to prevent hot spots, which may lead to skin burns.
- Do not use the electrode pads for more than 10 times, as connection between the electrodes and the skin deteriorates over time.
- The adhesive force of the electrodes depends on the skin properties, storage condition, and the number of applications. If your electrode pads are no longer fully sticking to the skin surface, replace them with new ones. Stick the electrode pads back onto the protective foil after use and store them in the storage bag to prevent them from drying out. This retains the adhesive force for a longer period.
- Before applying the electrode, it is recommended for users to wash and degrease the skin, and then dry it.
- Never remove the electrodes from the skin while the device is still on.
- Use only original electrodes recommended by the manufacturer. Other electrodes could cause injury.
- Everyone reacts differently to electrical nerve stimulation. There may be individual deviations in the correct placement of the electrodes. Contact your doctor to find out which placement is best for you.
- Do not use any adhesive electrodes with a size smaller than those the original manufacturer attached. Otherwise the current density may be too high and cause injuries.
- The size of the adhesive pads may not be changed, e.g. by clipping off parts.
- Make sure that the area where the pain comes from is completely covered with the electrode pads. For painful muscle groups, all affected muscles must be reached by the electrodes.

#### Notes on the TENS application

- If you feel the output intensity too strong, you can press „-“ button to decrease it;
- If you don't feel any discomfort during the treatment, we advise you to use the device until the session ends. Normally, the pain relief occurs after 5-10 mins of treatment;
- We recommend 1 to 2 treatments per day and one week as period of treatment;
- If pain relief is not achieved or the pain gets even worse after that period of treatment, please consult your doctor.

#### Notes on the EMS application

- Place the electrodes correctly as described in this manual;
- We recommend 1 to 2 treatments per day and one week as period of treatment;
- If you feel discomfort during treatment, you can either pause the session or decrease the intensity of the output.

#### Maintenance and cleaning

- To ensure a trouble-free and long-term functional safety of the device, the following points should be considered:
- Remove the electrodes from the device and clean the device with a soft, slightly damp cloth. In case of heavier dirt, you may also apply a mild detergent.
  - Protect the device from humidity. Do not hold the appliance under running water nor submerge it in water or other liquids.
  - Do not place the device on hot surfaces and do not expose it to direct sunlight.
  - Clean the surface of the electrode pads with a slightly damp cloth. Make sure that the device has been switched off before cleaning!
  - For hygienic reasons, each user should use his/her own set of electrodes.
  - Do not use aggressive substances or chemical cleaners to clean the device.
  - Make sure that no liquid penetrates into the device. Do not use the device until it has completely dried.
  - Never clean the device during operation. Before cleaning, make sure that the device is switched off and the batteries have been removed!

#### Maintenance

- The manufacturer can not be held responsible for damage caused by unauthorized repairs. If your device has a technical problem, contact the customer service.
- Do not try to do repairs on your own.
- Opening the device will void the warranty.
- Your device has been tested before sale and requires no calibration or regular maintenance. If you have any questions, please contact the customer service.

#### Troubleshooting

Problem	Possible reasons	Countermeasures
No display after replacing the batteries.	Batteries not inserted correctly or foreign matter in the battery compartment. Battery type may be wrong.	1. Check and clean the compartment. 2. Insert new batteries of the correct type. Pay attention to the correct polarity!
No or too weak stimulation	- Electrodes not correctly positioned on the skin - Connection to the device is not optimal - Used batteries - Too dry skin	1. Check that the electrodes have been correctly placed on the skin and that the cable connection to the device is optimal. 2. Replace the batteries. 3. Wipe your skin and the electrodes with a damp cotton cloth.
The skin tickles or burns during the application	- Application time too long. - Electrodes are not placed correctly on the skin or the contact is not optimal - The skin is hypersensitive	1. Apply once a day and shorten the period of use. 2. Check that the electrodes are correctly applied to the skin. 3. Wipe off your skin and electrodes with a damp cotton cloth. 4. For hypersensitive skin consult your doctor.
The device stops during the application	- Electrodes are not placed correctly on the skin or the contact is not optimal - Used batteries	1. Check that the electrodes are correctly applied to the skin. 2. Replace the batteries.

#### Storage

Turn off the device and remove the cables. Place the electrodes on the carrier foil and package them together with the device and the cables in their original packaging. Observe the storage conditions in the technical data! If you do not use the device for a long time, remove the batteries from the battery compartment.

#### Program tables

Mode	Part of the body	Frequency Hz	Impulse width µs	Therapy duration min.	Curve form
TENS	Neck	80-120	120-100	Default: 30 Adjustable: 5-90	Modulation
	Shoulder	80-100	100	Default: 30 Adjustable: 5-90	Modulation
	Arm	2-60	260-160	Default: 30 Adjustable: 5-90	Modulation
	Hand	100	100	Default: 30 Adjustable: 5-90	Continue
	Back	60/50/45/10 /50/35	200	Default: 30 Adjustable: 5-90	Modulation
	Abdomen	120	55	Default: 30 Adjustable: 5-90	Continue
	Hip	100	150	Default: 30 Adjustable: 5-90	Burst
	Leg	40/6/50	250	Default: 30 Adjustable: 5-90	Modulation
	Foot	80-120	100-120	Default: 30 Adjustable: 5-90	Modulation
	Joints	120	100-120	Default: 30 Adjustable: 5-90	Modulation
EMS	Neck	30	200	Default: 30 Adjustable: 5-90	Synchronous
	Shoulder	45	200	Default: 30 Adjustable: 5-90	Synchronous
	Arm	50	150	Default: 30 Adjustable: 5-90	Synchronous
	Hand	4	200	Default: 30 Adjustable: 5-90	Synchronous
	Back	60	200	Default: 30 Adjustable: 5-90	Synchronous
	Abdomen	20	200	Default: 30 Adjustable: 5-90	Synchronous
	Hip	30	150	Default: 30 Adjustable: 5-90	Synchronous
MESSAGE	Leg	80	200	Default: 30 Adjustable: 5-90	Synchronous
	Foot	20	200	Default: 30 Adjustable: 5-90	Synchronous
	Knead	28-44	120-250	30	Modulation
	Rub	25-79	120-250	30	Modulation
	Tap	49-97	100-240	30	Modulation

#### Disposal

This device may not be disposed of in the household garbage. Every consumer is required by law to dispose of electric or electronic equipment, regardless whether or not they contain hazardous substances at a collection point in his/her city or at the retail outlet so that they can be disposed of in an environmentally-friendly manner.

Remove the batteries before you dispose of the unit. Do not dispose of used batteries in the household garbage, instead dispose of them as special waste or at a battery collection point in a specialist outlet. Contact your local administration or vendor with respect to disposal.

#### Accessories

Cable set - article number 88359  
8 self-adhesive electrodes - article number 88349

#### Technical specifications

Name and model:	medisana 3-in-1 Electro Therapy Device TT 200 R-C4B
Model no.:	3 x 1.5V ; Micro batteries (AAA)
Power supply:	2
Channels:	Bi-phase square-wave pulse
Curve form:	Max. 120mA (at 500 Ohm load)
Output:	0 to 40 levels, adjustable
Intensity:	±20% for all parameters
Output precision:	TENS, EMS and MASSAGE
Treatment modes:	TENS: 10; EMS: 9; MASSAGE: 3
Number of programs:	55 - 260 µs, depending on the program
Impulse width:	2 - 120 Hz, depending on the program
Frequency:	5 - 90 minutes, adjustable, depending on the program after 1 minute
Application time:	+5°C to +40°C at 15% to 93% rel. humidity, atmospheric pressure from 700 hPa to 1060 hPa
Automatic switch-off:	-10°C to +55°C at 10% to 95% rel. humidity, atmospheric pressure from 700 hPa to 1060 hPa
Operating conditions:	109 x 54.5 x 23 mm
Storage conditions:	50 x 50 mm
Dimensions approx.:	76 g
Size of electrode pads approx.:	88347
Weight approx.:	4001690 88347 4
Article no.:	
EAN number:	

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

#### Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

- The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
  - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - Accessories which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**medisana GmbH**  
Carl-Schurz-Str. 2, 41460 NEUSS  
GERMANY

The service centre address is shown on the attached leaflet.

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